



AMG



AMG products are designed to work using hydraulic technology unlike no other catheter on the market. There is no longer a need to expose the female to a boar post heat detection. In fact, do not expose the female to a boar. I suggest breeding females while they are being fed. You simply seat the catheter like a conventional catheter in the cervix. Attach your bottle to the catheter. Pull slightly back on the catheter straightening the cervix folds. Place one hand on the catheter at the vulva to ensure the catheter stays in the desired position. Squeeze the semen bottle. The hydraulic pressure of the semen will deploy the membrane through the cervix into the uterus. The purple die represents the deployment of semen in the uterus. Total time can be less than one minute.

AMG Series catheters consistently provide up to 1.5 more live-born piglets, and 5%+ higher farrowing rates compared to what you are getting now. Therefore, ask yourself how much is a weaned piglet worth? How much do you make when selling market animals? And finally ask yourself how much does the AMG Catheter cost? Take whatever profit you normally make per animal, then subtract the cost of the catheter(s). Obviously, you will see a profit and the technology has proven to make you money!